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STELLA

STELLA II

Find this notice
at www.logo-silver.fr

Before using this product, please
read this manual carefully
to insure user safety.

LOGO SILVER
EUROPE

TABLE OF CONTENTS

General Information 3

Specifications and Intended Use 3

Standards and guidelines 3

Warnings 4-5

Your Rollator 6

Cleaning/Maintenance/Reassigning 6

Unfolding your Rollator 7

Brake Handles 7

Brake Adjustment 7

Climbing Small Obstacles 8

Handle Height Adjustment 8

Back Strap Adjustment 8

Sitting/Getting Up 9

Folding your Rollator 10

Attaching/Removing the Storage Bag 10

Cup Holder Installation 11

Cane Holder Installation 11

Using the Footrests (Optional equipment) 12

Using the Armrests (Optional equipment) 12

Slow-Down Brakes Adjustment (Optional equipment) 12

Technical Specs 13

Transport and storage 13

Limited Warranty 14

Beyond warranty cover 14

Disposal 14

Contact Us 14

GENERAL INFORMATION

The following instructions are an integral part of the delivery of this rollator. The user should have them readily available at all times. They must be kept with the rollator, should it be transferred to another user.

The manufacturer reserves the right to make changes relative to the information provided in these instructions as a result of technical developments and improvements.

This document may not be reprinted, translated and/or duplicated in any form whatsoever, even in part, unless prior written consent from the manufacturer.

For updates on these instructions, please contact the manufacturer or visit www.logo-silver.fr.

SPECIFICATIONS AND INTENDED USE (FOR INDOOR AND OUTDOOR USE)

The **STELLA** rollator is intended as a walking aid for people with reduced mobility who weigh up to **300 lb (136 kg)**.

To occasionally use this rollator as a transport chair, the optional footrests and armrests must be installed.

The rollator should be used only on firm, flat surfaces. It is not intended to be used for any other purpose such as a climbing aid or to carry heavy loads. Any such use can be dangerous!

STANDARDS AND GUIDELINES

All the safety information in this manual relates to the french legislation in force and the provisions of the European Union. In other countries, it is necessary to respect the corresponding legislations and provisions.

The STELLA was manufactured in accordance with current technical regulations for safe operation. The safety of the STELLA is certified by CE and its declaration of conformity.


The Logo Silver Europe company declares under its responsibility that its product meets the requirements of European regulation MDR 2017/745 relating to class I medical devices (according to annex VIII chapter 111, rule 13).

WARNINGS: READ CAREFULLY BEFORE USING!

- ⚠ Before sitting on the rollator, make sure the Stella is fully opened and the seat is completely down.
- ⚠ Before sitting on the rollator, make sure the lumbar support is well positioned horizontally to prevent falling backwards. Verify that both knobs are fully tightened at same height.
- ⚠ Before sitting on the rollator, make sure the wheel locks are engaged. In transport chair mode, before the user transfers into or out of the chair, the attendant must apply both rear wheel locks and make sure that the front casters are in the forward position. If the wheels rotate while locked, please take the rollator back to the dealer for adjustment. When the brakes/locks are released, the wheels should rotate freely.
- ⚠ Do not lean to one side, always keep your weight balanced on the rollator. Leaning forward, while using the Stella as a transport chair without proper assistance, can cause tipping.
- ⚠ In rollator mode, before sitting, put your hands on the hand grips and back-up until your legs touch the seat. In transport chair mode, before sitting, back-up until your legs touch the storage bag and use armrests to lower yourself down.
- ⚠ When getting up from the rollator in transport chair mode, assistance is ALWAYS required.
 - a. Attendant must apply wheel locks by pushing down on the brake handles.
 - b. First, remove feet from footrests and place them firmly on the ground.
 - c. Second, use armrests for support before attempting to stand up.
- ⚠ Do not hang bags or other items on hand grips – risk of tipping over!
- ⚠ Changing direction abruptly could make the rollator tip over.

- ⚠ Use this rollator on hard, flat surfaces only. Avoid use on uneven terrain such as grass, gravel and sand.
- ⚠ Do not use the rollator to go up or down escalators.
- ⚠ If excess weight is put onto the hand grips, the rollator could tip backwards.
- ⚠ When folding or unfolding the rollator, watch for possible pinch points.
- ⚠ When seated, the brakes should be applied.
- ⚠ Test the brake handles regularly to make sure the rollator brakes work properly.
- ⚠ Avoid exposing the rollator directly to the sun or extreme heat for an extended period of time as this can cause the rollator's parts to overheat and burn the user.
- ⚠ This rollator is not intended to be used as a seat or otherwise in a moving vehicle.
- ⚠ Do not lean forward while using as transport chair.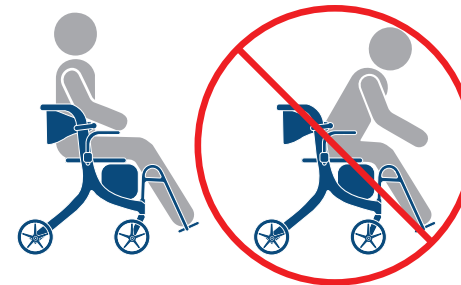

NOTE: You and your attendant should consult a qualified professional for the correct techniques on ramp negotiations.

WEIGHT CAPACITY: 300 LBS.

IF YOUR ROLLATOR BECOMES DEFECTIVE :

- ⚠ Do not try to repair it yourself, contact your authorized dealer immediately.

YOUR ROLLATOR

1. Handle height adjustment
2. Armrests (Optional)
3. Removable storage bag
4. Footrests (Optional)
5. Cane holder (not compatible with armrests)
6. Slow-down brakes (Optional)
7. Brake
8. Folding /unfolding seat strap
9. Cup holder
10. Back strap height adjustment
11. Brake handles
12. Ergonomic handgrips
13. Flip-back oversized back strap

CLEANING/MAINTENANCE/REASSIGNING

Cleaning

Clean and disinfect the rollator regularly. Use mild soapy water with no abrasive additives. Disinfect the rollator using commercially available products. Never use steam degreasers. Never use scouring or harmful cleaning agents.

Maintenance

The STELLA rollator requires little maintenance. If the braking function weakens, wheel movement becomes stiff or you notice any other functional impairment, please take the rollator to your authorized dealer for servicing. Never use any kind of lubricant on any parts of the rollator.

Reassigning to a new user

If you pass along your rollator to a new user, please remember to hand over all the technical documentation needed to ensure safe operation. In accordance with instructions, the rollator must be inspected, cleaned and returned to pristine condition by an authorized dealer before assigning to the new user.

USING AND ADJUSTING YOUR ROLLATOR

1. UNFOLDING YOUR ROLLATOR

Place the rollator in front of you in the direction of travel and apply the parking brake.

Pull the seat strap to release the locking mechanism (when available, the red tag indicates where to pinch and pull), press the left and right lateral tubes, on either side of the seat, down together, until the folding locking mechanism clicks into place.

2. BRAKE HANDLES

Pull to brake (FIG.1): Pull the brake handle up to slow down or stop the rollator.

Push to lock (FIG. 2): Push down on the brake handle to lock the rollator in its position.

3. BRAKE ADJUSTMENT

The brake system fitted on this rollator complies with the manufacturer's quality standards. The brake system has been adjusted to the best performance settings. To personalize your settings, follow instructions below.

You may need to adjust the brake system from time to time as it will wear out with normal use. To adjust the braking strength, push the screw housing downward and turn left or right (Figures 1 & 2).

4. CLIMBING SMALL OBSTACLES


Pressing your foot down on the extended frame (curb climbers) makes it easier to lift the front wheels to negotiate curbs.

5. HANDLE HEIGHT ADJUSTMENT (DEPENDENT ON MODELS)

Activate the brake to immobilize the rollator.

Model 1 (FIG.1): This adjustment should be done while standing. Press the gray adjustment button and move the push handle up/down until you obtain the required height. As a guide, the wrist should align with the handgrip. Once positioned, release the gray button.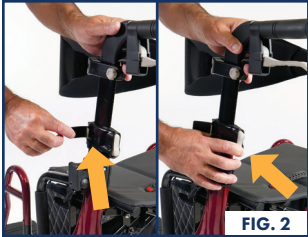

Model 2 (FIG.2): This adjustment should be done while standing. Open the tightening tab, press the gray adjustment button and move the push handle up/down until you obtain the required height. As a guide, the wrist should align with the handgrip. Once positioned, release the gray button and close the tightening tab.

6. BACK STRAP ADJUSTMENT

Strap height (FIG.1): Unscrew (yellow arrow) the gray knobs on both sides of the rollator and slide the back strap up/down on the extruded handle until the required height. The back strap should be well positioned on the back in the lumbar area. Once positioned, tighten the gray knobs (red arrow).

Strap rotation (FIG.2): The flip-back strap allows for use in either rollator mode or transport chair mode.

7. SITTING/GETTING UP

7.1 Rollator mode

Before sitting down, make sure the rollator is on a firm and level surface, the front castors are fully extended to the outside of the frame (in the forward position) and the brake handles are locked (pushed down).

Before getting up, make sure the brakes are still locked. Place your hands on the handgrips as a support while rising.

7.2 Transport chair mode

Attendant must be present in this mode. Before sitting down, the rollator must be on a firm and level surface and the attendant must:

- a) apply wheel locks by pushing down on brake handles;
- b) flip the back strap the opposite direction; and
- c) raise the armrests.
(See Section 13. using armrests)

The user must back-up slowly until legs touch the bag and position hands on armrests to sit down carefully. Footrest must be engaged after the user has sat down.
(See Section 12. using footrests)

Before getting up, make sure the brakes are still locked. First, remove feet from footrests and place them firmly on the ground. Footrests can be swung away toward the outside. Second, use armrests for support before attempting to rise.

8. FOLDING YOUR ROLLATOR

If footrests are installed on the rollator, flip up footplates and swing away footrests before folding.

Any content in the storage bag should be removed to ease the folding process of the rollator and prevent any damage to items.

Pull the seat strap (when available, the red tag indicates where to pinch and pull) and pull upward (FIG.1) until the rollator locks in folded position (FIG.2).

9. ATTACHING/REMOVING THE STORAGE BAG

To remove the storage bag, fold the rollator slightly (FIG.1) and slide bag out of receptacle, pulling on both ends (FIG.2).

To replace the bag, reverse above steps.

The SWL (safe working load) is 10 lb (4.5 kg).

10. CUP HOLDER INSTALLATION

Installing the cup holder (reversible left or right) requires no tools. Simply clip it below the handle on the opposite side of the cane holder (FIG.1).

Adjust the cup holder direction according to your preference (FIG.2).

The cup holder SWL (safe working load) is 3 lb (1.3 kg).

11. CANE HOLDER INSTALLATION (NOT COMPATIBLE WITH ARMRESTS)

Installing the cane holder (reversible left or right) requires no tools. Simply clip both cane holder's parts to the same side.

Clip the upper part underneath the handle (FIG.1).


Clip the bottom part above the brake or above the front wheel (FIG.2).

For proper cane alignment once clipped, both parts can be rotated to desired angle (FIG.3).

The cane holder SWL (safe working load) is 3 lb (1.3 kg)


12. USING THE FOOTRESTS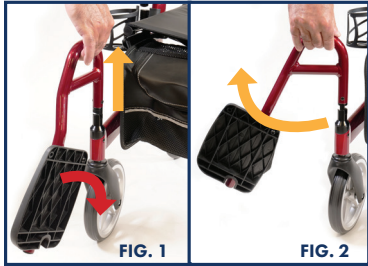

To engage: The attendant must pull the footrest upward, rotate forward until it locks into place and then flip the footplate down. Repeat on the other side. (FIG.1)

To swing away: The attendant must pull the footrest upward, rotate toward the outside and flip the footplate up. Repeat on the other side. (FIG.2)

13. USING THE ARMRESTS

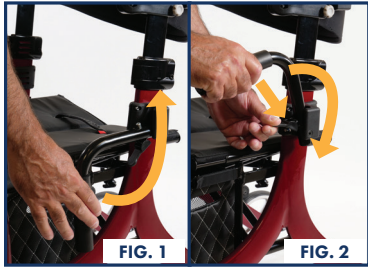To engage: The attendant must pull the armrest up from underneath until a click is heard. (FIG.1)

To clear away: The attendant must press release tab and push the armrest down. (FIG.2)

14. SLOW-DOWN BRAKES ADJUSTMENT

For a user with an uncontrolled gait, it is possible to slow the wheels down with the slow-down brake system.

To adjust the tension, turn the black knob clockwise to tighten (slower) or turn counter clockwise to loosen (faster) the tension. Perform this adjustment on both sides.

Slow-down brakes might need to be adjusted a few times on either side to obtain equal tension.

TECHNICAL SPECS

	STELLA 18	STELLA 20	STELLA 23	STELLA HD
Seat height	18" (455 mm)	20" (510 mm)	23" (585 mm)	20" (510 mm)
Seat width	17.5" (445 mm)	17.5" (445 mm)	17.5" (445 mm)	19.5" (495 mm)
Overall width	24" (610 mm)	24" (610 mm)	24" (610 mm)	26" (660 mm)
Handle height	29.5" - 34.5" (750 mm - 875 mm)	30.5" - 35.5" (775 mm - 900 mm)	33.5" - 38.5" (850 mm - 980 mm)	30.5" - 35.5" (775 mm - 900 mm)
Footrest length	15" - 17.5" (380 mm - 445 mm)	17" - 19.5" (430 mm - 495 mm)	20" - 22.5" (510 mm - 570 mm)	17" - 19.5" (430 mm - 495 mm)
Overall length STELLA	28.5" (725 mm)	28.5" (725 mm)	28.5" (725 mm)	28.5" (725 mm)
Overall length STELLA II	39.7" (1010 mm)	39.7" (1010 mm)	39.7" (1010 mm)	39.7" (1010 mm)
Weight capacity	300 lb (136 kg)	300 lb (136 kg)	300 lb (136 kg)	300 lb (136 kg)
Weight	14.5 lb (6.6 kg)	14.5 lb (6.6 kg)	15 lb (6.8 kg)	16 lb (7.3 kg)

NOTE - Some metric measurements are rounded to the nearest 5 mm.

TRANSPORT AND STORAGE

- The rollator may be transported folded or unfolded.
- The rollator must not be used as a seat in a vehicle.
- Your rollator must be stored in a dry place, sheltered from extreme temperatures and humidity.

LIMITED WARRANTY

The rollator comes with a 2 year warranty from the date of delivery, excluding corrosion, worn parts (tyres etc), abnormal use, non-compliance with the instructions for use. The batteries come with a 1 year guarantee.

- The warranty covers any defect in the construction or quality of the raw material and components, under normal conditions of use (excluding normal wear of the parts).
- The warranty is limited to the repair or free exchange of parts and subassemblies recognized as defective (parts, labor and transport), after inspection by an authorized representative.
- Warranty repairs must be authorised and performed by an approved supplier. You must not return parts without obtaining prior authorization.

BEYOND WARRANTY COVER

Accidental shocks or accidents, negligence, abnormal use, improper maintenance, modifications made without the authorization of the manufacturer, use not in accordance with the instructions in the user guide supplied with the product or which do not correspond to the product specifications, withdrawal / erasure of the serial number, cold storage or batteries left completely empty for a long period.

DISPOSAL

Respect the environmental protection regulations in your country. Out of use batteries are taken back by your dealer or Logo Silver Europe. The rollator can be returned to the Logo Silver Europe company for recycling.

ENVIRONMENTAL NOTICE

If you are disposing of the rollator, it is necessary to remove all the components and materials with respect to the environment and in accordance with the recycling and sorting of waste.

TO CONTACT US:

LOGO SILVER
EUROPE

Logo Silver Europe - 163 cours Berriat - 38000 Grenoble - France
Tél. +33(0)4 76 21 22 19 - contact@logo-silver.fr - www.logo-silver.fr